

DSSE

PRODUCT MANUAL



QUATTRO TREATMENT BED TB014

The Ferrari of Treatment Beds

Engineered with four precision electric motors, this top-of-the-line beauty bed delivers unmatched power, performance, and luxury. Designed for elite professionals, it combines cutting-edge functionality with sleek comfort—making it the ultimate companion for high-end salons and clinics where excellence is non-negotiable.

Introduction

This manual is intended solely as a reference for the operation of this specific model. DSSE assumes no responsibility or liability for any consequences resulting from its use for other purposes. All information contained in this manual is protected by copyright and may not be copied, reproduced, or translated, in whole or in part, without the prior written consent of our company.

Safety Instructions

- Always disconnect the power supply before changing electrical components or performing maintenance and cleaning on the equipment.
- The plug must be connected to a properly installed and grounded outlet to prevent the risk of fire or electric shock.
- Do not stand on the equipment.
- Keep children away from the equipment; place it out of their reach. Always adjust the backrest to its highest position when leaving the equipment unattended.
- Do not place the equipment in damp or wet areas such as bathrooms to avoid electric shock or malfunction. The remote control should also be kept away from water.
- Avoid using the equipment in environments where painting, coating, or oxygen provision is taking place.
- Unless supervised or instructed by qualified personnel, people with physical disabilities, sensory disorders, mental impairments, lack of experience or common sense, and children are not permitted to use the equipment.
- If the plug is damaged, contact your distributor, service center, or a qualified professional for repair.
- Ensure there are no objects within the equipment's movement range during operation.
- All maintenance must be performed by trained professionals only.

Precautions

- The equipment should be operated by experienced personnel familiar with its structure, performance, and instructions.
- Read this manual carefully before installation, operation, and adjustment. Keep it for future reference.
- Do not use for single-side or single-point loading.
- Follow safety operation rules. Do not overload. Ensure all safety equipment is reliable and address any hazards immediately.
- Regularly tighten bolts and nuts to avoid failure due to loosening or vibration.
- Do not disassemble parts without proper authorization.
- If faults occur or abnormal noises are heard, stop usage immediately and have the equipment inspected.
- Only resume use after proper troubleshooting.
- This bed uses a three-wire power cord and must be plugged into a grounded outlet.
- Observe all warning labels on the product.
- Press and hold the control button when adjusting the bed height until the desired position is reached.
- Motor operation time: 2 minutes ON, 18 minutes OFF to prevent overheating).
- Clean dust and stains using a neutral cleaning agent after each treatment.
- Keep hands, feet, and head clear of moving parts during adjustments.

INTUITIVE DESIGN, UNPARALLELED PERFORMANCE

Experience the perfect blend of comfort, functionality, and cutting-edge technology with our Quattro Treatment Bed.

Designed to meet the demands of modern professionals, this bed offers multiple adjustable features including an ergonomic backrest, seat cushion, and footrest, all easily controlled via intuitive hand and foot controllers.

Crafted with high-quality materials, it ensures durability and long-lasting performance. Its smooth, silent motorized adjustments ensure seamless transitions between positions, enhancing both client comfort and operator efficiency. Safety features such as a reset button and sturdy construction make it a reliable choice for any professional setting.

Pull-out headrest (13cm)

The pull-out headrest provides adjustable support for enhanced comfort, allowing you to extend or retract as needed to perfectly accommodate different clients and treatment positions.

Adjustable Backrest (0° - 85°)

Easily adjustable from 0° to 85°, the backrest allows you to customize the seating angle for optimal comfort and support.

Height Adjustment (60 - 98 cm)

Adjustable height for ergonomic comfort and ease of use.

Rotatable (±110°)

Smooth rotation up to 110° each way (from centre) for ultimate flexibility and ease.

Inbuilt Foot Controls

Built-in foot pedals allow hands-free, effortless operation for enhanced convenience.

Swiveling Armrests (0°-180°)

Designed for maximum comfort and flexibility, the swiveling armrests easily rotate to provide optimal support and allow effortless access during treatments or procedures.

Inclineable Seat (10°)

Enjoy personalized comfort with a seat that gently tilts up to 10°.

Inbuilt Side Buttons

Conveniently integrated controls on the side for easy and quick adjustments.

Adjustable Legrest (0°-85°)

Effortless adjustment for maximum comfort and support.

Pull-Out Footrest

Easily extends to support legs, enhancing comfort during use.

Subtle Ambient LED Lighting

Integrated LED lighting in four colours adds a sleek, modern touch to your treatment space.




HOW TO USE

Follow these instructions to operate the electric beauty bed safely and effectively. Ensure all users are familiar with the controls and functions before use.



Backrest Adjustment

Press to raise or lower the backrest

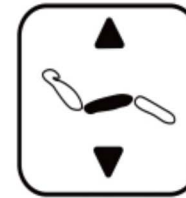

Incline Adjustment

to tilt the bed into the Trendelenburg position for inclined treatments



Height Adjustment

Press to raise or lower the bed between 58-96 cm



Legrest Adjustment

to raise or lower the legrest between 0° and 85°



Armrest Adjustment

Rotate armrests 180° for full access and flexibility.



Rotation Adjustment

Unlock the handle to rotate the chair 110° left or right. Push it back in to securely lock it in place.



Product Specifications

Length:	185cm
Width without/with Arms:	62cm/86cm
Height Range:	60-98cm
Backrest Adjustment Angle:	Up to 85°
Seat Tilt Adjustment Angle:	Up to 10°
Legrest Adjustment Angle:	Up to 85°
Net Weight:	84 kg
Gross Weight:	100 kg
Max. Load Capacity:	200 kg (seating position)
Power Supply:	AC 220-240V, 50Hz/60Hz
Max. Power Consumption:	300W




Effortless Button Operation

The Effortless Button is strategically positioned on both the backrest and the side of the seat cushion to provide convenient and immediate access. By simply pressing these buttons, you can effortlessly adjust the bed's position to suit your preferred working posture.


This intuitive control system allows for smooth, precise movements, minimizing physical strain and enabling you to focus fully on your professional treatments. The ergonomic design of the buttons ensures they respond to gentle touches, making adjustments quick and seamless without interrupting your workflow.

Specifically engineered with beauticians in mind, the Effortless Button enhances both comfort and efficiency during prolonged use, supporting better posture and reducing fatigue throughout your workday.

Cushion Cover Buttons

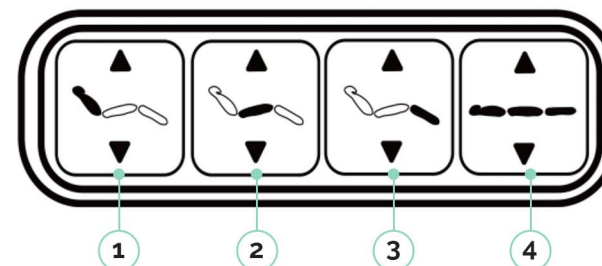

Backrest Cover Buttons

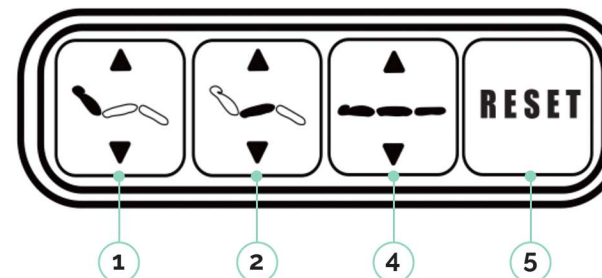


Hand Remote Operation

Please Note: All buttons on the device will flash during initial startup, indicating a locked state. Press the LOCK button to unlock. Once unlocked, the flashing will stop and the B light will remain steadily on to confirm the system is active.



- | | | | |
|----------|---------------------------------|----------|-------------------------------|
| A | Position Indicator Light | B | Power Indicator Light |
| C | Memory Function Indicator Light | D | Memory Button Indicator Light |

- 1** Press button to raise or lower the backrest
- 2** Press button to adjust the incline
- 3** Press button to raise or lower the legrest
- 4** Press button to raise or lower the height of the bed
- 5** Press and hold the button for 3 seconds to reset the chair. The backrest will fully recline, the chair height will lower to its minimum, the seat cushion will tilt to its maximum, and the leg rest will return to its lowest position automatically.
- 6** Press button to disable all functions. Press it again to unlock and reactivate the controls.
- 7** Press the button to turn on and change the ambient light color. Each press changes the color. After the last color, the light will turn off.
- 8** Press the button to adjust the ambient light brightness.
- 9** Hold the button for 3 seconds to activate the memory function. The Memory Indicator Light **C** will stay on for 3 seconds to confirm recording mode. Press **P1** to save the current position as Memory 1. Other number keys (e.g., P2, P3) work the same way to store additional positions.
- 10** Hold the button for 3 seconds to automatically move the chair to the saved position 1. Press any other button to stop the movement or to exit.
- 11** Hold the button for 3 seconds to automatically move the chair to the saved position 2. Press any other button to stop the movement or to exit.
- 12** Hold the button for 3 seconds to automatically move the chair to the saved position 3. Press any other button to stop the movement or to exit.

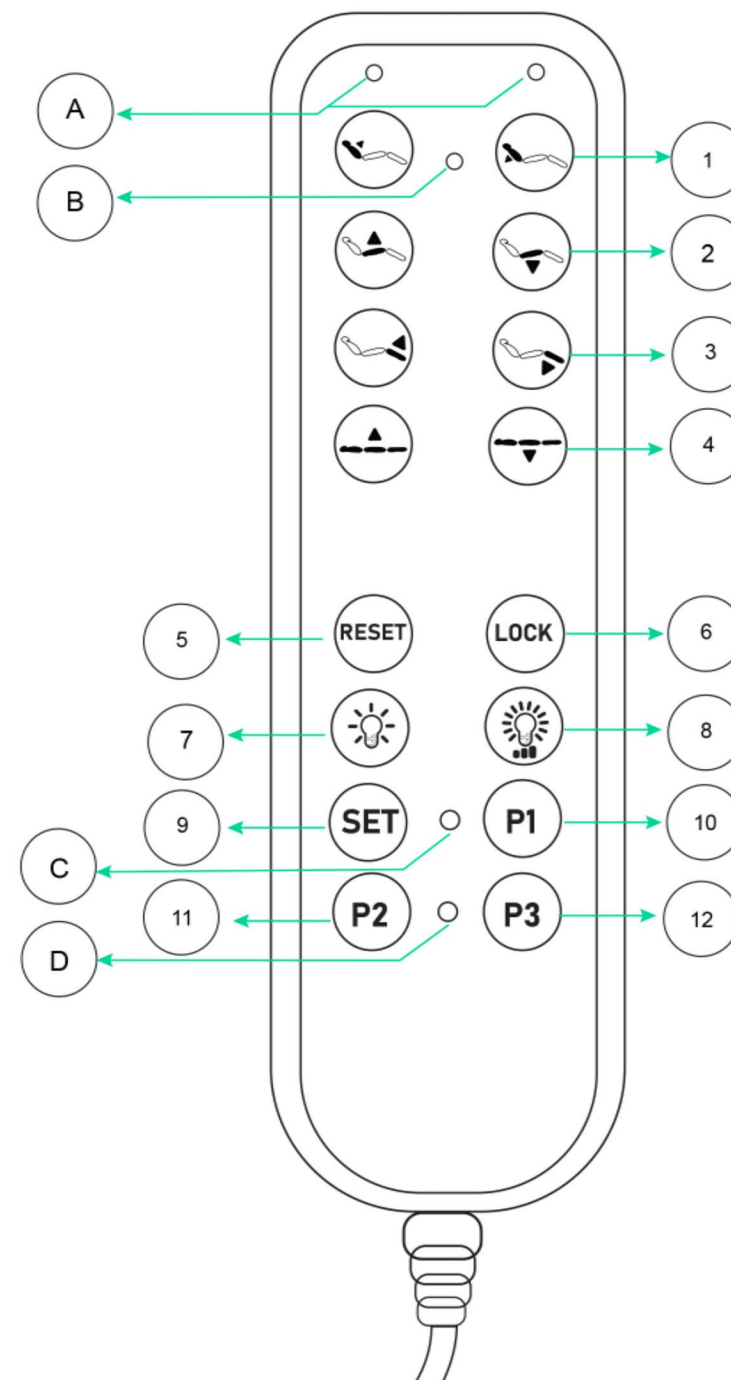


Headrest Assembly

To assemble the headrest, tighten the 4 screws as shown in the diagram below. All parts and screws are included in the bed packaging.

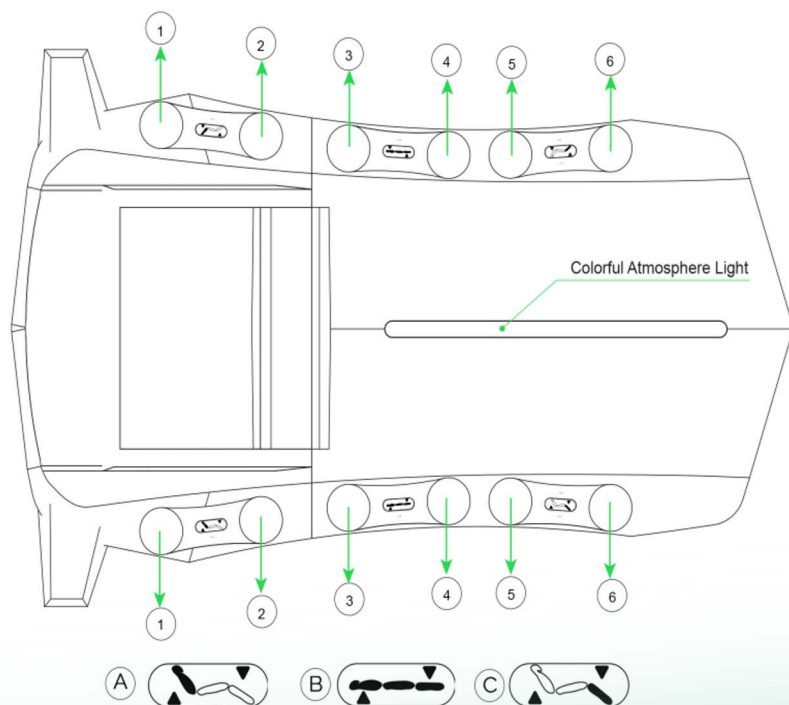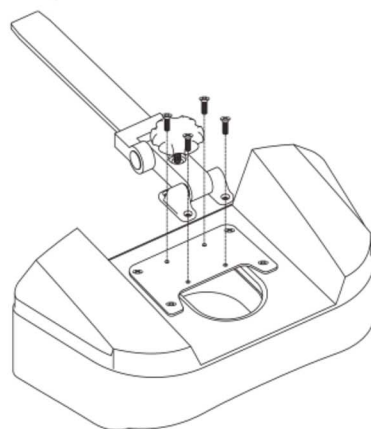

Switch Panel Instructions

1. Make sure the power cord and hand controller are connected in the correct direction and securely locked (follow the arrow indicators).
2. Turn on the power switch. The indicator light should stay on steadily.
3. If there is no response, switch off the power, open the fuse box, check the fuse, and replace it if needed.

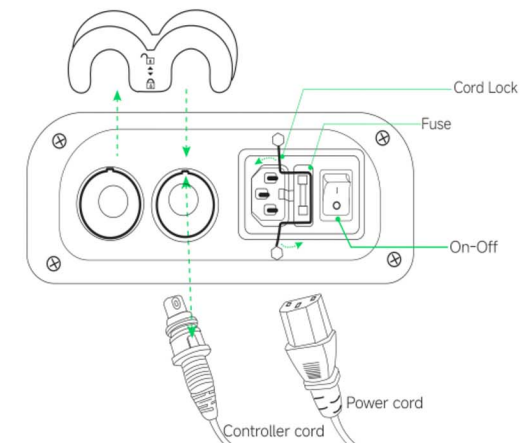

Inbuilt Foot Pedals

- A** Step on the foot pedal to adjust the backrest angle. Press **1** to raise the backrest and press **2** to lower it.
- B** Step on the foot pedal to control raising or lowering the height. Press **3** to raise the height or press **4** to lower it.
- C** Step on the foot pedal to control raising or lowering the legrest. Press **5** to raise the height or press **6** to lower it.

Upholstery Care Guide

As a professional esthetician, investing in a high-quality electric facial bed can significantly enhance the comfort and effectiveness of your treatments. To maintain that level of performance and ensure long-term reliability, regular maintenance and proper care are essential. Like any professional equipment, your facial bed requires routine attention to stay in optimal working condition—ensuring safety, smooth functionality, and a consistently elevated experience for your clients.

General Care Tips

Maintain a stable temperature:

- Cold can lead to vinyl cracking
- Heat may cause permanent stretching

Keep out of direct sunlight for long periods to prevent fading and damage. Regular housekeeping helps preserve comfort and appearance.



Cleaning Instructions

Clean frequently to remove body oils that can harden vinyl

Use a colorfast cloth dampened with lukewarm water and mild, alkaline-free soap

Avoid soaking the material

Wipe off any soap with fresh water and a clean cloth

Dry thoroughly with a soft, dry cloth

For tougher grime, gently scrub with a soft brush



Avoid Using These Products

- Hot or boiling water
- Alcohol or hydrocarbon-based cleaners
- Acetone
- Bleach or harsh detergents
- Furniture polish
- Abrasive powders, pastes, or steel wool
- Silicone or petroleum-based products
- Solvent-based or abrasive cleaning agents

Routine Inspection

Perform periodic checks on:

- Hinges • Fasteners • Structural components

Ensure your chair remains safe, stable, and fully functional.

Do not dispose of old or defective appliances with household waste. Please use designated public collection points for proper disposal. This manual is for reference only. Specifications may change without prior notice.